

GURIN

ULTRASONIC MASSAGER

INSTRUCTION MANUAL

MODEL : PU-110



Relieve muscular spasm / Relax, ease joint stiffness
Improve body circulation / Increase elasticity of collagen tissue

www.gurinproducts.com

FDA CE 0434

Contents

It is recommended that you read this entire manual thoroughly before using the device.

GENERAL INFORMATION.....	1
CONDITIONS TREATED.....	2
SAFETY.....	3
PRECAUTION.....	4
SPECIFICATIONS.....	5
GENERAL DESCRIPTION.....	6
SKIN PREPARATION.....	7
OPERATING INSTRUCTION.....	7
DO'S AND DON'TS.....	8
SERVICE AND REPAIR.....	9
WARRANTY.....	9



General Information

Our portable Ultrasonic Massager is an ultrasound device that can produce one million vibrations per second. This causes the tissues to compress and expand at the same frequency of ultrasound and this effect is called micro massage. Micro massage, stimulates the tissues and improves the metabolism of cell membrane and leads to the generation of frictional heat.

It helps circulation and muscle relaxation during the process and promotes healthy care for deep tissue. The ultrasound has an automatic timer which is preset to operate for 10 minutes on the treatment area, after which time it will shut off.

Ultrasound cannot transmit in the air, if the treatment head does not completely contact with the skin, the result would not meet your expectation. We recommend to apply ultrasound gels to conduct the wave energy to the skin.

1

To massage with Ultrasound for 10 to 15 minutes every day, could activate the cells and revive tissue fibers. The effect is obvious for relieving pain and accelerating recovery. Ultrasonic Massager excites the cells and strengthens tissue fibers, effectively preventing the aging process and recession.

Conditions Treated

- Back pain
- Chronic pain
- Golfer's elbow
- Muscle spasm
- Post exercise recovery
- Strains and sprains
- Tennis elbow
- Tendonitis
- Arthritis and more...

2

Safety

- The Ultrasonic Massager should not be used for the treatment of malignancies or in the region where malignant tumours exist.
- This is not a toy for children.
- Do not use on people with poor arterial circulation.
- Do not use over an area where bleeding or infections exist.
- Do not use over the eyes, skull, mouth, nose or vagina.
- Do not use on the abdomen if you are pregnant.
- Do not use for patients with demand type cardiac pacemakers
- Do not use over the carotid sinus nerves or arteries and laryngeal or pharyngeal muscles.

Precautions

- Special care must be given to patients with sensory losses. Areas that are temporarily numb or to a part of the body which has no feelings.
- Avoid bony prominences or bones close to the surface of the skin.
- When using ultrasound, keep the sound head moving while maintaining contact with the skin.
- Do not immerse in water or other solvent.
- Do not use over metallic implants, especially prostheses with a cement-matrix.
- Do not use the Ultrasonic Massager without using a conductive gel.

Specifications

Power Source:

AC adapter, 110 V AC (13.5 V DC) or

AC adapter, 220 V AC (13.5 V DC)

Frequency: 1 Mhz \pm 5%

Output Power: 0.5 watt \pm 20%

Switch: On/Off

Treatment area: 3.8cm²

Times Setting: 10 minutes \pm 10%, Auto off

Ultrasound Gel: Water soluble

Dimensions: 50 x 45 x 190 mm

Weight: 200 gm

Product includes

Ultrasonic Massager x 1

AC Adaptor x 1

Gel 60 ml x 1


Instruction Manual x 1

Package Box x 1

5

General Description

- The main unit: Easy to handle and manipulate.
- The connector.
- ON/OFF switch: switch button, simple ON and OFF, a built-in time setting of 10 minutes, auto disconnected power.
- 110 V AC (or 220 V AC) Adapter
- Gel: completely neutral water soluble gel.
- Treatment head: 3.8 cm²

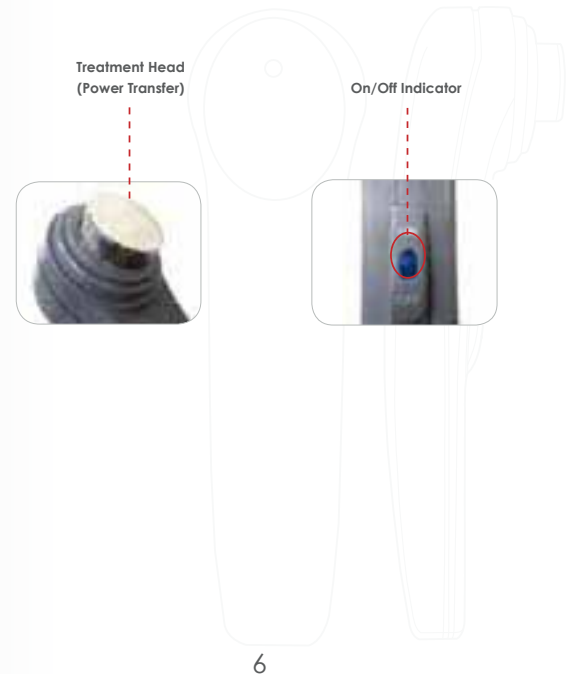

Skin Preparation

Clean skin before and after use.

Precautions

- Connect power wire to wire socket.
- Connect to power source.
- Apply suitable amount of gel to skin at target area (a circle with 2cm diameter).
- Switch on the power. If power signal light is on, start to use massager.
- DO NOT stop moving treatment head while in use.
- The power automatically turns off in 10 minutes or, if desired, the power can be turned off sooner by pushing the ON/OFF switch to OFF.
- After use, wipe excess gel off the treatment head with a damp cloth. Disinfect treatment head with antiseptic wipe. Do not immerse in water or solvent.
- After using the device, clean skin with water to remove gel.

- When power shuts off, leave the unit off for 30 minutes. After every treatment, disconnect AC adaptor from power source.
- Do not impact the treatment head or drop treatment head on the floor.

DO'S and DON'TS

DO

- Do follow instructions.
- Do keep the treatment head clean.
- Do clean cables with water and mild soap.
- Do not use rubbing alcohol or any other solvent.

DON'T

- Do not damage your Ultrasonic Massager by bumping, dropping, or rough use.
- Do not immerse in water or other solvents.
- Do not pull or kink cables.
- Do not store below 32F (0°C) or over 122F (50°C)

Service And Repair

To obtain service, call your dealer for instructions. You may need to send in your complete unit with the problem stated in writing for repair.

Warranty

We warrant each new Ultrasonic Massager unit for ONE (1) year from defects in materials and workmanship. This warranty only applies to the original purchaser. This warranty does not cover damage caused by abuse, or accidental damage which results in the failure of this unit. Failure to follow operating instructions, damage by natural force, dismantled or disassembly will void this warranty.

Product Name: Ultrasonic Massager

Model No: PU-110

Product serial number: _____

Name: _____

Date of purchase: _____

Distributor: _____



GURIN

