

WARNING

Please do not put this scale on slippery floor to prevent slip.

SPECIFICATIONS

4 High precision strain gauge sensors

Capacity: 5-180KG (11-400LB)

Division: 100G/ 0.2LB

Size: 310*325*31.5mm

Auto on/ Auto off

Overload and low battery indication

Unit: KG/LB/ST

Power: 4*AAA batteries

Work temperature: 10-50 C

Storage temperature: -10-50 C

BATTERY

Open the battery cover,


input 4*AAA batteries



OPERATION

STEP 1

Put the scale on a hard and flat floor
(Avoid carpet or soft surface).

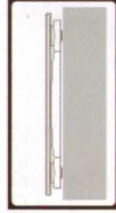

STEP 2

Step on, please evenly step on the scale.

Keep body balance and share the weight to feet. After several seconds, the LCD will display the locked weight data.




AUTOMATIC SWITCH OFF

The scale will automatically power off after 10 seconds without any operation.

CHANGE UNIT

Press the unit switch button at the bottom of the scale to choose unit "KG", "LB" or "ST".

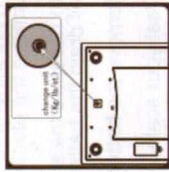

INDICATOR - WRONG INDICATION

When scale is overloaded, the LCD will display EEEE. Please move from the scale and avoid it is damage.



LOW POWER INDICATOR

When the LCD displays "Lo", the scale is low power. Please replace all batteries.

